# Xerthra™

## PRP kit

### Opis

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Data wydania: 06/2024 Rev. 02/2024

**BIOVICO**  
MEDICAL BIOTECH

## Instructions for use of the procedure pack

# Xerthra™

## PRP kit

### Description

The procedure pack designed for isolation and separation of platelet-rich plasma (PRP) from patient's own blood. Isolated autologous PRP is intended for local injections to improve the regenerative properties of cells and tissues by providing a large number of growth factors to specific areas.

### Procedure pack consists of the following medical devices:

- ① Xerthra™, *Biovico* (separation device), 1 pc.
- ② Venofix® A, 21G, *Nipro Corporation* (butterfly needle), 1 pc.
- ③ Microlance™ 3, 21G x 1½", *BD* (needle), 3 pcs.
- ④ dicoNEX Single use syringe, 3-part 20 ml, *Zarys*, 1 pc.
- ⑤ dicoNEX Single use syringe (with Luer-lock), 3-part 10 ml, *Zarys*, 1 pc.
- ⑥ dicoNEX Single use syringe (with Luer-lock), 3-part 5 ml, *Zarys*, 1 pc.
- ⑦ dicoNEX Single use syringe, 3-part 2 ml, *Zarys*, 1 pc.
- ⑧ dispoFINE 27G (0.40 mm x 19 mm), *Zarys* (needle), 1 pc.

### Cautions

To be used exclusively by qualified, trained personnel. Before using the procedure pack, please read the instructions for use. The procedure pack consists of disposable medical devices. Do not reuse. Do not use any of the medical devices in case of damage of its direct package. Medical devices are sterile only during their shelf life indicated on the package. Store the procedure pack at a temperature between +5°C and +25°C. Protect from direct sunlight, high temperature and high humidity. Keep out of the reach of children. Wear personal protective equipment when using the procedure pack. Ensure that no one is exposed to direct contact with collected blood. Place all used pack parts in a medical waste container and utilize them. Local symptoms such as pain, redness and swelling may occur at the injection site.

### Contraindications

Children, pregnant or nursing women should not be treated with PRP as there are no clinical data available on therapy in those groups of patients.

**NOTE: Use only freshly drawn blood for plasma isolation. PRP must be administered immediately after isolation.**

### Preparation before use

1. Remove Xerthra™ ① from the package, position it vertically and mark with the patient's code or name to avoid confusion.
2. Using the needle ③, withdraw 1.5 ml of anticoagulant into the 20 ml syringe ④.
3. Using the butterfly needle ②, collect the patient's blood into syringe containing anticoagulant. Fill the syringe up to 15 ml.
4. Mix gently by inverting the syringe.
5. Mark the syringe containing the collected blood with the patient's code or name.

**NOTE: To avoid contamination of the isolated PRP, perform the entire isolation procedure with caution to sterility.**

### Instructions for use

1. Check if the bottom screw cap of Xerthra™ ① is tightened correctly. Transfer carefully collected blood with anticoagulant using the needle ③ into Xerthra™ ① placed vertically on a flat surface, pushing the needle in the center of the top cap.
2. Open the centrifuge cover, place Xerthra™ ① and properly prepared counterweights inside.
3. Centrifuge for **5 minutes at 1700 xg** (RCF).
4. Open the cover and carefully take out Xerthra™ ① with the centrifuged plasma. Hold Xerthra™ ① upright, do not shake it in order to avoid mixing of the separated fractions.
5. Replace the bottom screw cap of Xerthra™ ① with the controller's screw cap and then fasten the controller's bolt to Xerthra™ ①.
6. Turn the bottom part of the controller's bolt counterclockwise, until the plasma reaches the last ring in the top part of Xerthra™ ①.
7. Open the top cap of Xerthra™ ①.

**NOTE: Each time before connecting the syringe to Xerthra™ ①, check if the syringe plunger is moving properly.**

8. Depending on the desired volume of PRP, follow the selected procedure:  
**1 ml of PRP**  
Attach the 10 ml syringe ⑤ in place of the top cap to collect platelet-poor plasma (PPP). Rotate the controller's bolt until the red blood cells (RBC) fraction reaches the line marked as '1' on Xerthra™ ①. Then, to withdraw 1 ml of PRP, connect the 2 ml syringe ⑦ to the top cap and turn the controller's bolt until the RBC fraction reaches the last ring in the top part of Xerthra™ ①.  
**2 ml of PRP**  
Attach the 10 ml syringe ⑤ in place of the top cap to collect platelet-poor plasma (PPP). Rotate the controller's bolt until the red blood cells (RBC) fraction reaches the line marked as '2' on Xerthra™ ①. Then, to withdraw 2 ml of PRP, connect the 2 ml syringe ⑦ to the top cap and turn the controller's bolt until the RBC fraction reaches the last ring in the top part of Xerthra™ ①.  
**3 ml of PRP**  
Attach the 10 ml syringe ⑤ in place of the top cap to collect platelet-poor plasma (PPP). Rotate the controller's bolt until the red blood cells (RBC) fraction reaches the line marked as '3' on Xerthra™ ①. Then, to withdraw 3 ml of PRP, connect the 5 ml syringe ⑥ to the top cap and turn the controller's bolt until the RBC fraction reaches the last ring in the top part of Xerthra™ ①.
9. Disconnect the syringe with ready-to-use PRP and attach the needle ③ or ⑧ to the syringe, depending on required procedure.
10. Mark the PRP syringe with the patient's code or name to avoid confusion.

### Organization assembling procedure pack:

**Biovico Sp. z o.o.**  
Hutnicza 15 B  
81-061 Gdynia, Poland, EU  
+48 58 660 44 88  
office@biovico.com  
www.biovico.com

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